


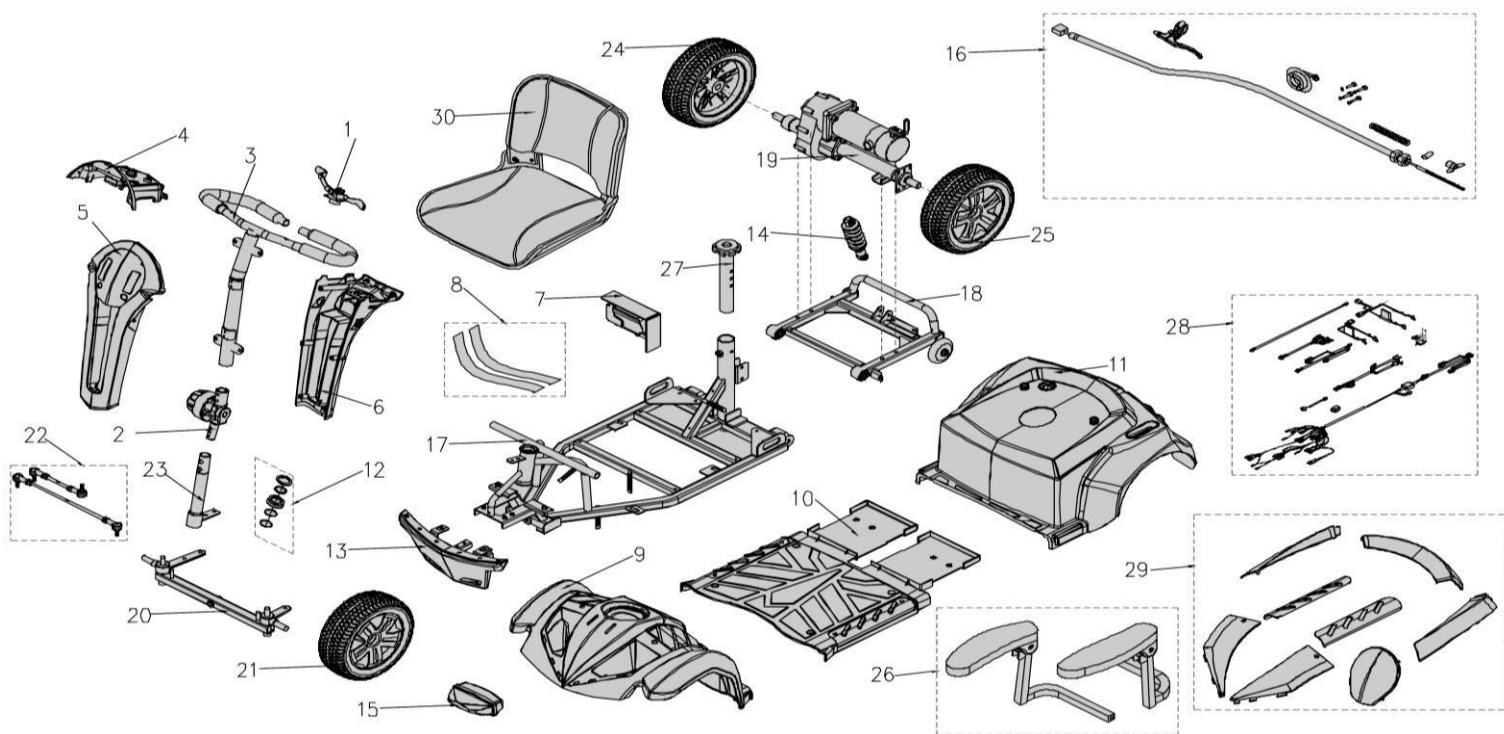

No.	Code	Description
1	S12-067-00100	Speed Dial Ass'y (Envoy)
2	S02-067-00100	Tiller Swivel Ass'y (Envoy)
3	S02-067-00200	Delta Tiller Ass'y (Envoy)
4	S08-067-00100	Cover, Handle Bar Ass'y (Envoy)
5	S08-067-00300	Shroud, Tiller Front (Envoy)
6	S08-067-00400	Shroud, Tiller Rear (Envoy)
7	S10-067-00100	PG S-Drive 70A Controller
8	S34-067-00100	Velcro Strip Set, Battery (Envoy)
9	S08-067-00200	Shroud, Front (Envoy)
10	S08-067-00500	Shroud, Footrest Ass'y (Envoy)
11	S08-067-00600	Shroud, Rear (Envoy)
12	S02-001-00601	Regular Bearing Ball Set (Envoy)
13	S01-067-00100	Front Bumper Ass'y (Envoy)
14	S04-067-00100	Shock Absorber, 250lb (Envoy)
15	S13-067-00100	Headlight Ass'y (Envoy)

No.	Code	Description
17	S01-067-00200	Frame, Front Section (Envoy)
18	S01-067-00300	Frame, Rear Section (Envoy)
19	S09-067-00100	Transaxle Ass'y, 4mph (Envoy)
20	S03-039-00400	Front Fork Ass'y (Envoy)
21	S06-067-00100	Wheel 300-5 Pneumatic Envoy
22	S03-033-00400	Connecting Rod Ass'y (Envoy)
23	S03-067-00100	L Type Steering Axle (Envoy)
24	S06-067-00200	Rear Wheel, brake shoe, Envoy
26	S07-067-00100	Armrest Ass'y (Envoy)
27	S07-067-00300	Seat Post Set (Envoy)
28	S18-067-00200	Wire Harness Ass'y (Envoy)
29	S08-067-01718	Shroud Set, Red (Envoy)
	S08-067-01754	Shroud Set, Blue (Envoy)
	S08-067-01777	Shroud Set, Graphite G. (Envoy)
	S08-067-01791	Shroud Set, White (Envoy)
30	S07-067-00100	Seat Assy, Envoy 4 (no armrests)

Separate Components Available:

From the Motor & Transaxle Ass'y (S09-062-00100):

C09-067-00200 (Transaxle, Envoy 4)
C09-067-00400 (EM Brake, Envoy 4)
C09-001-10101 (Motor, 350W)
C09-005-00601 (Carbon Brush, 7x17mm)
9357005005035 (Keyway)

Other Electrical Parts

C18-001-01501 (Ignition Switch)
C12-001-10202 (Charger Socket)
C11-045-00102 (Battery Gauge)
C11-001-00700 (Speed Pot)
C16-001-10500 (Headlamp Switch)

C16-001-10500 (Horn Switch)

Wiring Looms

C18-067-00101 (Main loom, Envoy 4)

C16-022-00402 (30A Circuit Breaker)

C18-001-10302 (Battery Cable – Fused)

C18-001-10402 (Battery Cable – Unfused)

C18-067-00400 (Power Cable to Controller)