



## **Pill Puncher**



**VM926A**

## **Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for deciding to purchase the Pill Puncher from Aidapt. The Pill Puncher is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

## **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

This Pill Puncher is a useful aid for those with limited dexterity or weakness in the hands and arms. It assists the user in releasing tablets from a blister pack or foilbacked sheet of pills. It can be used on a flat surface with one hand, or simply held in and operated by one hand, whichever is most comfortable for the user.

## **INSTRUCTIONS FOR USE**

The Pill Puncher is simple to use, we recommend that you place it on a flat surface for ease of use:

- Place the Pill Puncher onto a flat surface or hold in your hand, with the sprung arm uppermost.
- Position the tablet blister pack below the arm, with one tablet on the recessed area.
- Press the arm of the Pill Puncher down firmly, with the raised punch section directly above the tablet.
- The tablet will be punched free from the blister into the storage compartment below.

## **CLEANING**

Clean your Pill Puncher using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.